



Wilcare Guide to Handset Functions for Rise and Recline Chairs

Single motor handset

- 1 Enables the chair to move to a rise position
- 2 Returns the chair to a seated position, followed by the footrest being raised. On a tilt-in-space chair the seat and back will start to recline together. By keeping the button depressed the back will then recline



Dual/3 motor max handset

- 1 Moves the footboard to a raised position
- 2 Returns the footboard to a closed position
- 3 Reclines the back
- 4 Returns the back to a upright position
- 5 Enables the chair to move to a rise position
- 6 Returns the chair to a seated position



3 motor tilt in space handset - glamorgan

- 1 Enables The chair to move to a rise position
- 2 Returns chair to seated position. Moves the footboard to a raised position
- 3 Brings the back to an upright position
- 4 Reclines the back
- 5 Returns from tilt in space position
- 6 Create the tilt in space position (whilst in recline mode)



3 motor, extra max and bariatric handset

- 1 Moves the footboard to a raised position. Returns chair to seated position when chair has been in rise position
- 2 Returns the footboard to a closed position. Keeping this button pressed will move the chair into a rise position
- 3 Reclines the back
- 4 Returns the back to a upright position



2 motor 4th position handset

- 1 Enables the chair to move to a rise position
- 2 Moves the footboard to a raised position followed by the seat going into a tilt position
- 3 Reclines the back
- 4 Returns the back to a upright position



2 motor portable handset

- 1 Moves the footboard to a raised position
- 2 Returns the footboard to a closed position
- 3 Reclines the back
- 4 Returns the back to a upright position
- 5 Raises the footboard and reclines the back simultaneously
- 6 Returns the back and footboard to a upright and closed position respectively

